



Above: Shown with Sign Holder, MP-047 (sold separately)
Below: Using shelf area for miscellaneous medical supplies



Product Description:

- Protection System
- Holds a variety of gowns, three boxes of gloves, one box of earloop-style face masks, one box of shield-style face masks, and built-in storage area (for a variety of medical supplies, fourth box of gloves, or one box of N-95 face masks)
- Keyholes for wall mounting or may be used with a BOWMAN® door or wall hanger
- Made of quartz powder-coated aluminum

Primary Area for Product Usage:

- All patient care areas requiring personal protective equipment and isolation supplies, including oncology clinics, dialysis centers and clinic exam areas

Dispenses:

- Top left section: Gowns folded up to 14.4"W x 4.5"D
- Top right section: One box of shield-style face masks up to 8.1"W x 9.6"H x 4.5"D
- Middle right section: One box of earloop-style face masks up to 8.2"W x 5.0"H x 4.5"D
- Four bottom sections: Four glove boxes up to 11.3"W x 5.4"H x 4.0"D, or three glove boxes and miscellaneous medical supplies in bottom right

Accessories (sold separately):

- Sign Holder MP-047
- BOWMAN® door and wall hangers: MB-450, MB-500, MB-510, MB-600, and MB-601
- Clip-On Caddy Dispensers: MP-015 and MP-085


Package Specifications:

- 26"L x 6"W x 32"H (66.0 cm x 15.2 cm x 81.3 cm)
- 12 lbs (5.4 kg) - approximated

Product Specifications (overall external dimensions):


- 22.75"W x 25.63"H x 4.58"D (57.8 cm x 65.1 cm x 11.6 cm)
- 7.6 lbs (3.4 kg) - approximated

Mounting Specifications:

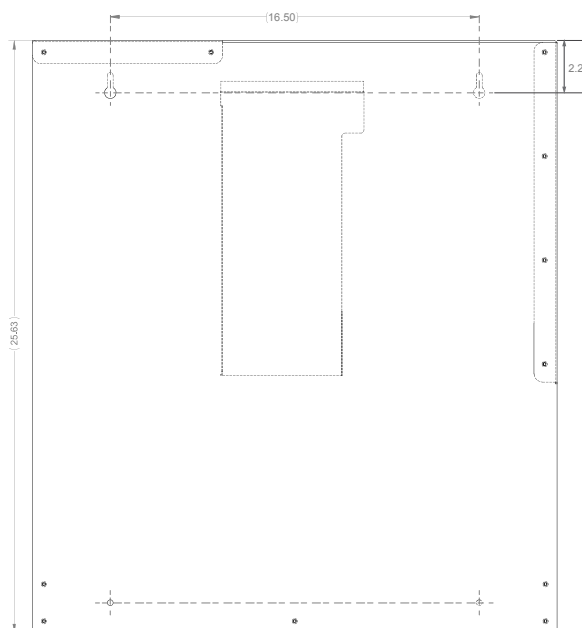





1

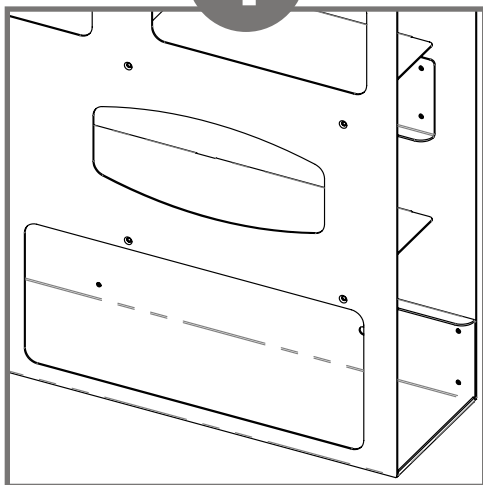

Open section at lower right can be used to store a variety of boxed or loose medical supplies.

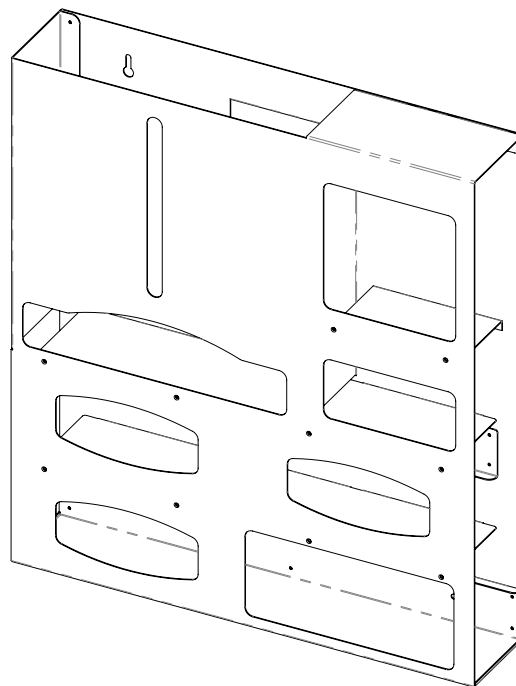


2

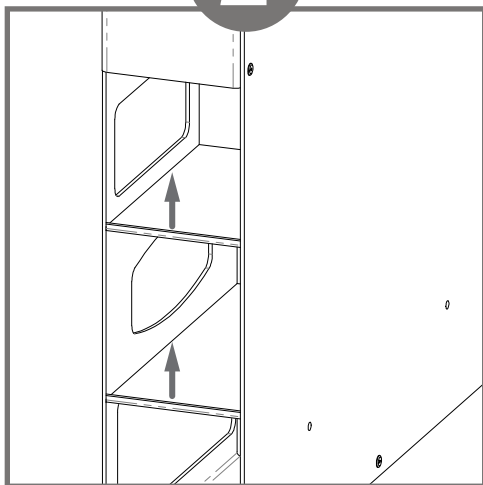


The slight upward curve on the side is designed to hold product in the dispenser. To remove product, simply lift slightly to clear the edge and pull box out.

3

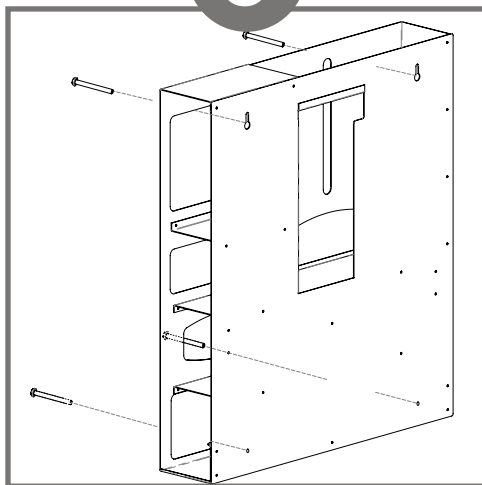

Dispenser should be mounted securely to a wall through keyholes in back with screws approved and provided by building maintenance.

4

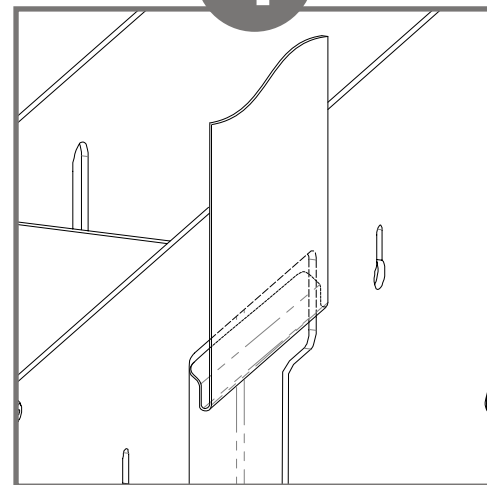

Alternatively, dispenser can be attached to a surface by hanging on a BOWMAN® door or wall hanger (sold separately) through the cut-out on the back of the dispenser.

Parts included:

- Protection system

Tools required:

- No tools required to assemble system

Advantage

Designed for Long-Term Use

Our aluminum dispensers and stands are coated with polyester triglycidyl isocyanurate (TGIC) powders, making them highly corrosion resistant. They are lightweight, attractive, functional and provide years of service.

Major Benefits

Durable

Our powder-coated aluminum dispensers resist corrosion, chipping, and flaking. This material will provide long-lasting products that hold up well, even in tough environments.

Cleaning

The semi-gloss powder-coated finish resists mild chemicals and cleaning agents, such as Windex or glass cleaners. Standard hospital cleaning supplies are approved for use on our powder-coated products or can be cleaned with a damp cloth or mild soap and water. Scotch-Brite, steel wool or any abrasive products should not be used.

Strength

Powder-coated aluminum is ideal for dispensers used in a healthcare or industrial settings. It is impact resistant and is not affected by extremes in temperature.

Appearance

The clean, bright appearance of powder-coated aluminum looks attractive and provides a professional image in all environments.

